

ivNow is an innovative fluid warmer that is convenient, easy to use, and will save you money.

- Simply place an IV bag in the ivNow.
- Warm IV fluids to 40°C/104°F in less than 30 minutes.
- Automatically turns on when a bag is placed on the cradle, and turns off when the bag is removed.
- Install it almost anywhere: on counters, on the wall, or on mobile equipment poles.
- Eliminate time spent setting up and handling in-line intravenous warming disposables.
- Eliminate all in-line warming disposable costs.

See ivNow in action at
www.ivnow.com/video



ENTHERMICS
Medical Systems

W164 N9221 Water Street
P.O. Box 443
Menomonee Falls, WI 53052-0443 U.S.A.

Tel: 262-251-8356 | 800-TO-B-WARM
Fax: 262-251-7067 | 800-329-8744

generalinfo@enthermics.com

www.enthermics.com | www.ivnow.com
ISO 13485:2003 Certified

#E123 09/11

Enthermics
MEDICAL SYSTEMS

IV Fluid Warmers



ivNow[®]

U.S. Patent WO 2010/030746
Canadian Patent 2692640

www.ivnow.com
ISO 13485:2003 Certified
262-251-8356 | 800-TO-B-WARM

ivNow[®] Fluid warmers

ivNow modular fluid warmers put warmed fluid at your fingertips where you need it, when you need it. ivNow fluid warmers are small, convenient units that can be placed in a number of different combinations and environments. Countertop, wall mounted and mobile units are all available depending on your requirements.

ivNow quickly warms and maintains safe temperatures of intravenous and irrigation fluids. The specially contoured warming surface cradles 0.5-, 1-, 2- and 3-liter solution bags. Each cavity is controlled by an individual control and L.E.D. display. A sensor in the heating plate detects the presence of a bag and engages the heating mechanism to quickly warm the fluid. Two temperature sensors continuously read the temperature of the fluid bag to engage the heater as necessary to maintain the fluid temperature within $+0/-2^{\circ}\text{C}$ ($+0/-3^{\circ}\text{F}$) of the set point temperature.

ivNow warmers ensure your fluids are kept at the proper temperature and will always be immediately available at or close to body temperature rather than room temperature without the added expense and set-up time required with in-line warming systems.



Available Configurations

ivNow-1 (one bag capacity) can be placed directly on a countertop, mounted on an optional countertop pedestal stand, mounted on a mobile equipment pole stand, or mounted on a wall using brackets*.

ivNow-2 (two bag capacity) can be placed directly on a countertop, mounted on a countertop pedestal stand, or mounted on a wall using mounting brackets*.

ivNow-3 (three bag capacity) must be mounted on a countertop pedestal stand, a heavy duty mobile equipment pole stand, or on a wall using mounting brackets*.

ivNow-4, -5, and -6 (four, five and six bag capacity, respectively) are mounted on a wall using mounting brackets. (Mounting hardware sold separately.)



Note: Multiple cavity units are available from the factory in the configurations above. Individual ivNow units cannot be stacked in the field. Three-liter bag capacity available on ivNow-1, ivNow-2, & ivNow-3 with an additional adapter.

To learn more, contact your Enthermics representative at (800) TO-B-WARM or visit us at www.ivnow.com.